

THE LIGHT CAP®

The Light Cap® was developed to help the cranial and upper cervical areas, which are two of the most important areas of the horse's body. Twenty-eight diodes are placed over specific muscle trigger points, acupuncture and other therapeutic points for stimulation.

Some of the benefits include the following:

Frontal sinuses - helps relieve pressure and drain sinuses.

Temporal mandibular joint - in humans, this area is known to cause headaches and create other subtle problems in distant areas of the body.

Poll area - helps relax and calm the horse, loosens tight muscles and allows the nerves to fire, relieving pain and tension. This area is also known to cause problems in other areas such as the back.

Directions for use

1. Place the Light Cap over the horse's head with the ears through the ear holes (the Velcro piece goes in front.)
2. Attach Velcro at top of "V" to form "Cap."
3. Pull the forelock through opening over behind the Velcro. This helps insure that the lights are not obstructed.
4. The halter strap should be placed above the battery pack to help prevent the horse from shaking it off.
5. Included is a Velcro Strap which will help hold the Light Cap in place while you put the halter over the top, and hold the corners down gently over the pole area.
6. Please note: the unit may be damaged if the horse is left unattended. It has been reported that some get so used to it their owners can leave it on during transporting, however use your judgment. Damage is not covered under warranty.



To replace the three (3) AA batteries: Remove the two Phillips head screws from the control box. Remove the cover and old AA batteries. Insert three (3) fresh AA batteries, noting proper orientation. Replace the control box cover (noting correct position) and screws.

For more information on our full line of BioScan™ products please contact an Independent Sales Representative, visit our website www.bioscanlight.com/, phone 505-797-1076, email: jjordan@omnilightled.com or mail us at OmniLight, Inc. dba BioScan PO Box 26716 Albuquerque NM 87125