

## The LightPatch™

The LightPatch has 32 Red LEDs (light emitting diodes) concentrated into a 2¼” by 5” treatment area; the pad measures 4 ½” by 8”. This very versatile product was designed to be used anywhere on muscles, tendons, ligaments, etc. The unit has a 30” cord that separates the pad from the Battery-Pack/Controller. The cord makes it convenient to distance the controller from the pad where necessary or the controller can be attached to the Velcro™ backing on the pad.



### Directions for use:

1. The unit comes with three AA batteries, ready to use. The recessed on and off buttons are located on front panel, adjacent to an indicator light just to the left of the “b” in bioscan.
2. The indicator light will illuminate GREEN when the unit is operating.
3. The indicator light will turn RED when the batteries need to be replaced. Note: the LEDs on the pad may still illuminate when the indicator light is RED, however the unit will not be operating at optimum level. Once the indicator light turns red, replace the batteries to avoid battery leakage or corrosion that could damage the unit.
4. Replace the batteries by removing the two screws on the back. Remove the old batteries and install three fresh AA batteries with proper orientation. Replace the cover, securing the two screws.
5. The unit will automatically shut off at the end of 30 minutes.

For more information on our full line of BioScan™ products please contact an Independent Sales Representative, visit our website [www.bioscanlight.com/](http://www.bioscanlight.com/), phone 505-797-1076, or mail us at OmniLight, Inc. dba BioScan PO Box 26716 Albuquerque, NM 87125